
MEDICAL PATIENT SCALES

CLASS  APPROVED

INSTRUCTIONS FOR USE



UNITS G & H, RICH INDUSTRIAL ESTATE, AVIS WAY,
NEWHAVEN, EAST SUSSEX BN9 0DU
TEL: 01273 515310 FAX: 01273 516750



Comply with EC Regulations - effective 1st January 2003
EC Type Approval No. UK 2668

INTRODUCTION

The WeighCare range of medical patient scales weighs up to:

WeighCare Model	Capacity	Accuracy
'H' - Hoist Scale	200kg	100g
'B' - Baby Scale	6kg	2g
'BT' - Baby Toddler Scale	60kg	20g
'C' - Chair Scale	230kg	100g
'P' - Person Scale	230kg	100g
'D' - Doctors Scale	235kg	100g
'PH' - Person Scale High Capacity	300kg	200g

SCALE FUNCTIONS

ON / ZERO / OFF

Press once to switch the scale on. When in operation, a single press will zero the scale. To switch off, press and hold down for 3 seconds.

BODY MASS INDEX (BMI)

To enter this mode, press and hold down the BMI key, with the patient on the scale. It only functions above 35kg. The display invites you to enter your patient's height in metres. Do so by using the up and down arrows. Then press the BMI key once and the display will show your patients BMI.

POWER SUPPLY

'P', 'D', 'PH', 'B' & 'BT'

The unit is powered by the supplied WeighCare mains adaptor when plugged into the jack connection on the side of the display unit. As standby, the unit can operate from one internal replaceable battery - a PP3 type, 9 volt. The system switches itself off after 2 minutes of non-use to conserve power.

'H' & 'C'

The unit is powered by an internal LA battery. Charging is by means of the supplied WeighCARE charging unit.

Low battery indication is shown when all decimal points in the weight display are shown.

The scale switches off after 2 minutes to conserve power.